

I-neb Cleaning Guide

A simplified guide to **cleaning and maintaining your I-neb AAD Device**
All images below are for illustrative purposes only

Regular maintenance

Your device will need to be given a **quick clean** regularly to ensure it keeps operating effectively. Find out how to do this below:

Top of handset

Use an **antiseptic wipe** (can be purchased online) to wipe away any moisture around the **horn and sensor port cover** (below)



Body of handset

Whenever the body of the I-neb has liquid spilled on it, wipe with a **clean damp cloth**





Routine Wash

~ 5 mins

Do this after every treatment

- 1 After each use place the **mouthpiece, medication chamber, chamber lid and drug guide (all the parts that have been in contact with the medication)** in the **washing basket**. Do not place the I-neb handset under water.
- 2 Submerge the basket containing the above parts in a bowl of **warm soapy water (add three drops of liquid detergent)**.
- 3 Move the **basket back and forth** in the water for around 30 seconds
- 4 **Rinse the washing basket** containing the parts in **soft, distilled or filtered water**. If you have hard water, rinse with water that has been boiled and allowed to cool.
- 5 **Shake off excess water** and leave parts to air dry.



~ 15-20mins

Weekly Wash

Do this **every week** or if you've **stored the I-neb for a long period of time** before using again

- 1 Once per week place the **mouthpiece, medication chamber, chamber lid and drug guide (all the parts that have been in contact with the medication)** in the **washing basket**. Do not place the I-neb body under water.
- 2 **Place and submerge the basket in a pan of soft, distilled or filtered water with three drops of liquid detergent**.
- 3 Then **bring the pan of water to a boil**. Boil for **10 minutes**.
- 4 Allow the water to cool and **remove the basket (be careful not to touch the hot water)**.
- 5 **Rinse the basket containing the parts with soft distilled or filtered water**. If you live in a hard water area, rinse the basket with water that has passed through a filter or that has been boiled and allowed to cool.
- 6 **Shake off excess water** and leave the parts to air dry.



Frequently Asked Questions (FAQs)

Regular Maintenance FAQs:

Q. Can I wash the handset under running water

A. No, the handset contains electronics and cannot be cleaned under running water

Q. Can I use kitchen roll or kitchen cloth to clean around the top of the handset

A. It is advisable to use **antiseptic wipes**

Q. Can I place the handset in a steriliser

A. No the handset contains electronics and cannot be placed in a steam steriliser

Daily & Weekly Washing FAQs:

Q. Can I clean the parts in a steam steriliser?

A. No, the daily and weekly cleaning recommendations for the drug guide and chamber assembly need to be followed to ensure the I-neb continues to work properly



Use the **Wash Basket** for all daily & weekly washing


Q. Can I wash the parts under running water?

A. No, place the **mouthpiece, drug guide and medication chamber assembly in the cleaning basket provided**. This will protect all of the parts during washing.

Q. Can I place the washing basket in the dishwasher?

A. No, the daily and weekly cleaning recommendations for the drug guide and chamber assembly need to be followed to ensure the I-neb continues to work properly

Visit **Promixinsupport.com** for further information and resources

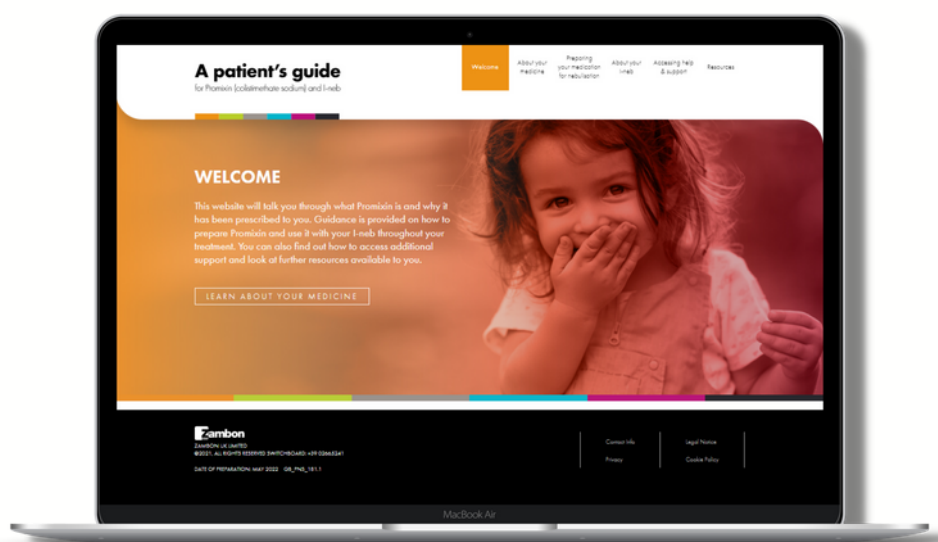

Or **scan the QR code** below for the full user guide :

